



**US
DENTAL
DEPOT**

US DENTAL DEPOT

6555 NW 9th Avenue suite 212
Fort Lauderdale 33309
Florida USA

USA - 1 866 625 5558 – 1 954 874 6325

Colombia 57 1 489 9051

Peru 51 1 707 3444

Venezuela 58 212 729 1639

Argentina 54 11 5258 7148 Cba 54 351 569 1409

www.USDENTALDEPOT.com



ECCO Instant Mouthguards

Assorted Item # 822710 Clear Item # 822700

ECCO Instant Mouth Guards are Fast, economical and cast to construct. They are excellent for orthodontic patients. They firmly adapt to teeth and does not lock on wire. They are also easily trimmed with scissors. Use for Immediate relief of TMJ

One minute mouthguard technique:

1. Check tray in mouth for fit. Any excess can be trimmed with scissors
2. Drop tray into boiling water for exactly 14 seconds. The tray will contract into a "C" shape
3. Quickly remove tray and carefully shake off all hot water. Press firmly over teeth and into vestibule



**US
DENTAL
DEPOT**

US DENTAL DEPOT

6555 NW 9th Avenue suite 212
Fort Lauderdale 33309
Florida USA

USA - 1 1866 625 5558 – 1 954 874 6325

Colombia 57 1 489 9051

Peru 51 1 707 3444

Venezuela 58 212 729 1639

Argentina 54 11 5258 7148 Cba 54 351 569 1409

www.USDENTALDEPOT.com

4. Have patient bite into tray. Hold for 3 to 5 minutes until set.



Step #1



Step #2



Step #3



Step #4

WWW.USDENTALDEPOT.COM