



TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA

ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401

Manage Software Setup

1 - Insert the USB flash drive. Setup should start automatically.



2- Open folder to view files

3- Double click in CAMERA SOFTWARE AND DRIVER

VERY IMPORTANT:

INSTALL FIRST:

TD-DRIVER FOR 106944 and 105800

4- When you finish installing, go back to install: SOFTWARE 106944 and 105800.



TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA


ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401
marcos@usdentaldepot.com
6555 NW Powerline Rd suite # 211-212 - Fort Lauderdale 33309 USA



TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA

ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401

5- You are going to have these two icons on your Desktop:

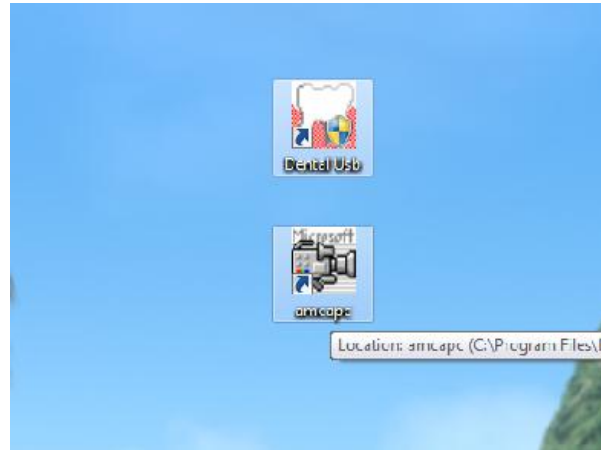

-Your wireless transmitters must be on the correct transmission channels

-If you are using on your notebook, change the button on the transmitter, if you are using in your computer make sure it is in the correct place.

-Don't use USB hubs, to connect your usb device. You must connect direct on the computer to work properly.



TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA

ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401
marcos@usdentaldepot.com
6555 NW Powerline Rd suite # 211-212 - Fort Lauderdale 33309 USA



TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA

ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401

6- Click on Dental USB and this window is going to pop-up



7- Click twice in the middle of the screen, this window will appear.



TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA

ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401
marcos@usdentaldepot.com
6555 NW Powerline Rd suite # 211-212 - Fort Lauderdale 33309 USA



No.	Name	Identity card	Tel


8- Input one number (you can choose any number). Input your name, and click Add.

Double click in your name, and this screen is going to appear:



TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA

ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401

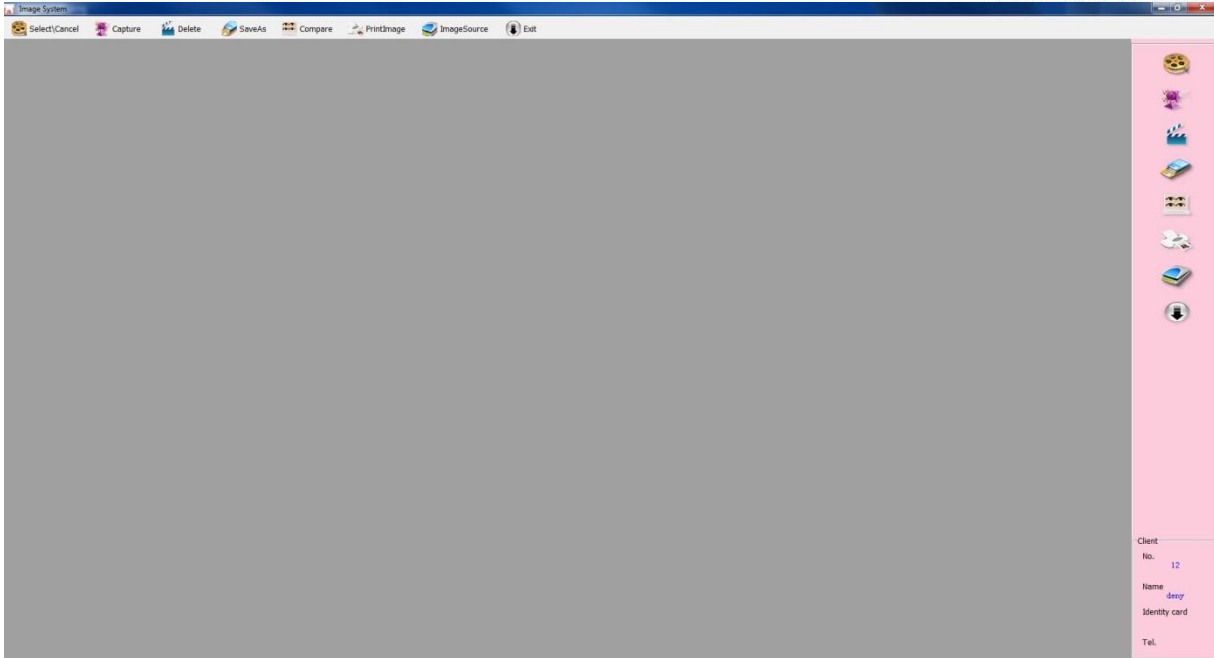

9- Click on Icon: CAPTURE

Make sure:



TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA

ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401
marcos@usdentaldepot.com
6555 NW Powerline Rd suite # 211-212 - Fort Lauderdale 33309 USA




TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA

ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401

Video Device: TPC (WDM)

Video Source: AUX

Capture Size: 640x480

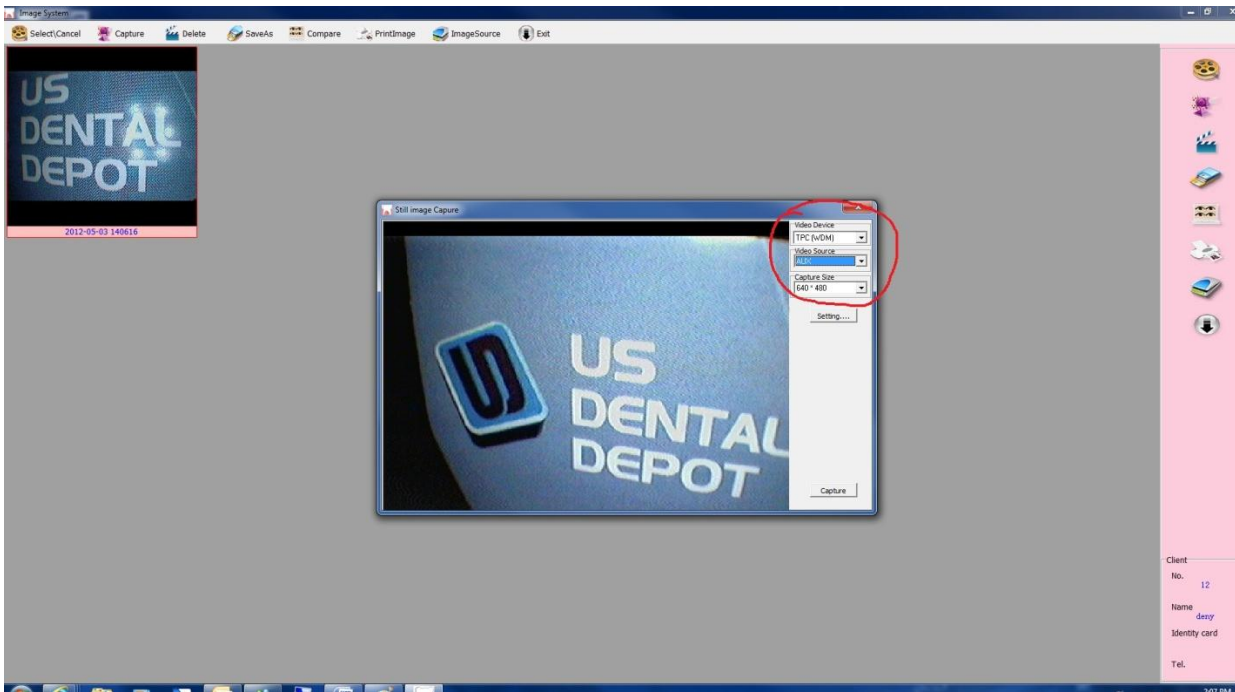

THIS MANUAL IS MENT TO BE USED TOGETHER WITH THE USER GUIDE



TO ORDER 1 954 874 6325
Fort Lauderdale FLORIDA USA

ARGENTINA (54) 11 5258 7148
CORDOBA (54) (351) 569-1409
COLOMBIA (57) 1 489 9051
PERU (51) 1 705 53 71
BRAZIL (55) 11 3711 9401
marcos@usdentaldepot.com
6555 NW Powerline Rd suite # 211-212 - Fort Lauderdale 33309 USA